InFloCone™ V

Patent Pending / Patente Pendiente / En attente de brevet

PART / PARTE / PRODUIT #1459050

Alliance
TECH MEDICAL

4053 Action Hwy, Suite 103
PO Box 6024 Granbury, TX 76049
800-848-8923
www.alliancetechnical.com

Product Description and Intended Use:

InFloCone™ is a disposable mouthpiece for use with Metered Dose Inhalers designed to provide guidance to the "Open Mouth" method for inhaling aerosol medications.

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Instructions for use: / Instrucciones para uso – / Mode d'emploi:

1 Read all instructions carefully before using your InFloCone.

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2 Follow all manufacturers' instructions for the preparation of the pMDI inhaler before use.

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3 Take InFloCone mouthpiece out of the plastic bag.

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4 Remove any cap that may be covering the end of your pMDI inhaler.

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5 Note: The InFloCone oval mouthpiece (has rolled edge) can be placed in the mouth vertically or horizontally.

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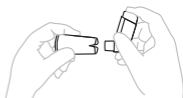
- 6 Attach clasping end (has tabs and vents) to inhaler according to your directional preference while holding the pMDI upright. Note: Do not turn the InFloCone while it is on the inhaler as this may damage the clasps.**

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
- 7 Exhale completely, then place rolled edge of oval end gently between teeth, close lips. Do not bite down on mouthpiece.**

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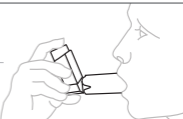- 8 After starting a gentle inhalation (30 to 60 lpm) press down on the pMDI canister to release the medication while continuing to inhale gently and completely.**

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- 9 InFloCone™ is a disposable / non-sterile mouthpiece designed for single patient use only.**

El InFloCone es una boquilla plástica desechable/no-esterilizada diseñada para un solo uso; InFloCone est un embout jetable / non-stérile conçu pour l'usage unique seulement.

Visit www.inflocone.com or scan this QR code for instruction video.

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Cleaning Instructions: / Instrucciones De Limpieza – / Instructions de nettoyage:
InFloCone can be cleaned in warm soapy water, rinsed and left to air dry.

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CAUTION: / ATENCION – / ATTENTION:

Federal Law restricts the mouthpiece to sale by or on the order of a physician or clinician licensed to order such use.

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